

**Update Notice:**  
**Disposable Face Mask Application Instructions**

*Due to a printing error, please refer to the correct instructions below*

**Product Code: 600003**  
**Disposable Face Mask - Level 2 Submicron**

TGA 296802



Hold the mask with the mouldable white strip at the top as this is the nosepiece, with the white lining facing you (coloured lining to face externally).



Apply mask to your nose and mouth.  
Pinch white strip to conform to your nose.



Use the nosepiece to hold mask to face,  
secure two ear loops around your ears and  
expand pleats to cover your chin.



Ensure the nosepiece fits tightly around the  
nose and mask sits under chin for a secure  
facial fit.

**Contact us for any clarifications:**

PH: 03 9554 2100

E: [info@ultrahealthmedical.com.au](mailto:info@ultrahealthmedical.com.au)

[www.ultrahealthproducts.com.au](http://www.ultrahealthproducts.com.au)