

## Surgical Mask with Ear Ties Application Instructions

ARTG 334569



Hold the mask with the mouldable white strip at the top as this is the nosepiece, with the white lining facing you (coloured lining to face externally).



Apply mask to your nose and mouth.  
Pinch white strip to conform to your nose.



Secure the ties around the back of the neck and head. Expand pleats to create void placing the bottom of the mask under your chin.



Ensure the nosepiece fits tightly around the nose and mask sits under chin for a secure facial fit.

### **Contact us for any clarifications:**

PH: 03 9554 2100

E: [info@ultrahealthmedical.com.au](mailto:info@ultrahealthmedical.com.au)

[www.ultrahealthproducts.com.au](http://www.ultrahealthproducts.com.au)